# 735mm

148mm

148mm

105mm

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**EN Instructions for Use: WE PEDI**  
**Product Description**  
WE PEDI is an adaptable, ready to use bandage for the foot that covers the foot up to the ankle. The product is intended to be used on intact skin. The inside layer of the laminate consists of a non-normal fabric which causes low friction (slipper forces) to the skin. WE PEDI can support the medical treatment of abnormally shaped feet (e.g. diabetic feet). This is realised using a two layered non-normal fabric with incorporated elastic parts. The hook and loop fastener system is repositionable. WE PEDI can be used on the right or left foot.

**Medical Indications**  
Adapted, ready to use bandage for the foot: the product also supports the medical treatment of abnormally shaped feet (e.g. diabetic foot). For use as fixation bandage for universal use (also for advanced). The bandage should retain its adhesion on the foot depending on the size is possible. Use as protective bandage of wound pad material or plaster.

**Method of application (recommended) as directed by a doctor**  
**APPLICATION STEPS**  
1. Open the fasteners and put the WE PEDI over the toes.  
2. Pull WE PEDI back over the foot.  
3. Stretch the WE PEDI to the length of the foot and pull it up over the heel.  
4. Close all fasteners according to instructions.

**Additional information**  
It is recommended to put on WE PEDI sitting or lying down. Select the appropriate size of WE PEDI according to the size chart.

	M	L	XL
Shoe Size (EU)	37 - 42	41 - 46	45 - 49
Foot Length in cm	20 - 26	25 - 32	28 - 35

The following factors may require a different size of WE PEDI:  
• extremely voluminous feet  
• normal feet  
• amputations  
• extra layers of primary/secondary layers

WE PEDI is correctly applied when no cavities form, there are no gaps or wrinkles, and the material does not apply pressure/compression.

Only use WE PEDI as described in the instructions for use. Any other use is considered as not intended and may lead to material damage and personal injury.

When using WE PEDI, make sure that the bandage is not applied too tightly, does not stretch the foot, and has sufficient elasticity on the foot.

If there is any uncertainty about the use or if the health condition worsens, the use should be immediately discontinued, and medical advice should be obtained. When working with WE PEDI, it is recommended to wear adequate footwear (e.g. shoes, surgical shoes).

Please note: while using this product, have any reason to assume that a serious event has occurred, please report the event to the manufacturer and your national authority.

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